


COMFORT  
**DUETT**  
A quality product from Sweden

**TELEVISION KIT**

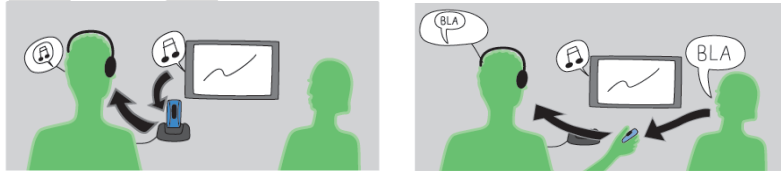

*...continued from page 2*

*Using Comfort Duett to amplify the sound of an electronic source:*  
Place the Comfort Duett in the base unit. Slide the mode selector on the Comfort Duett to the top position "MIC/TV". Adjust the volume to a suitable level.

When the Comfort Duett is lifted from the base unit it will automatically switch to MIC mode and work as a communicator. It is therefore easy to switch between amplification of the television and amplification of a conversation, as shown in the pictures above.

When using Comfort Duett in the base unit amplifying a sound source, it is also being charged at the same time.

It is possible to connect the cables from the television/other sound source directly into Comfort Duett communicator unit. Connect plug (B) into the socket at the top right corner of Comfort Duett. Please note that the Comfort Duett will not be charged when not placed in the base unit.

*You can download a copy of the Comfort Duett User Manual from our homepage. [www.comfortaudio.com](http://www.comfortaudio.com)*



Comfort Audio Inc., Customer Service, 5161 Overland Ave,  
Culver City, CA 90230, Phone: (310) 945 3048

COMFORT  
**DUETT**  
A quality product from Sweden

**TELEVISION KIT**

*The television kit makes it possible to amplify the sound from a variety of sources like TV, CD-player, radio, etc with the Comfort Duett. The television kit is easily connected via the base unit.*



USTV01

The television kit comes with 2 different cables. The type of audio out jack on your TV determines which cable to use.

*Installation guide:*

Place the base unit on a table and connect the power supply cord to a working electrical outlet and to the DC power jack on the left side of the base unit. To wall mount the base unit, see the Comfort Duett User Manual.

Connect plug (B) on the 30 feet cable into the jack on the right side of the base unit. We recommended you place the cable along the walls from the television; see the picture on the opposite page.



There are 2 different ways to connect the television to Comfort Duett. Try first the recommended way, alternative 1).

1) If the television (or other sound source) has a "Line out" in the back or front, connect the two plugs (A) on the 30 feet cable into that jack. When possible, match colors. The sound level from the television/sound source is generally fixed and is adjusted on the Comfort Duett.

2) If the sound source is lacking the "Line out" jack, then connect the two female plugs on short cable (C) with the two male plugs (A) on the 30 feet cable. Connect the male plug on cable (C) into the headphone jack on the television/other sound source.

Note: It is common that the loud speakers will be disconnected (no sound) on the television/other sound source when a plug is connected to the headphone jack. The sound level of the sound source will in most cases affect the sound level delivered from the Comfort Duett.

*Continued on the back page...*

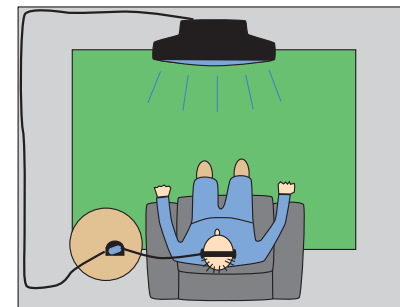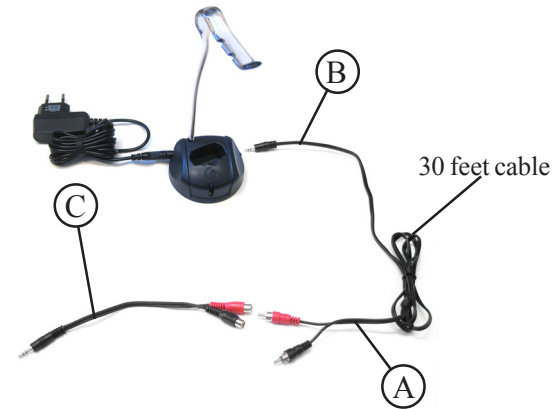

*You may place the cable along the walls or under the carpet for a more permanent installation.*

*Troubleshooting when Comfort Duett does not amplify the television:*

- a)** Make sure that the cable(s) is plugged into the television correctly.
- b)** Make sure that the cable from the television is plugged into the base unit correctly.
- c)** Make sure that the mode selector is in the top position and the green LED indicator on the base unit is lit.
- d)** Make sure that Comfort Duett is placed correctly in the base unit.
- e)** Make sure that the headphones are plugged in correctly.
- f)** Make sure that the volume is correctly adjusted.
- g)** Make sure that the maximum volume control is correctly adjusted.

*For more information, please see the Comfort Duett User Manual.*